



### Package Insert for DuraDerm®

**Intended Use and Product Description:** DuraDerm® is indicated for providing a covering over minor wounds and scrapes that are clean and dry. It provides a clear, elastomeric, non-odorous film for covering minor wounds, cuts, abrasions, burns and scrapes. The film protects the wound against water, dirt and germs and is not intended as a treatment or cure for all types of wounds. The elastic properties of the film help protect in difficult body regions where flexing, bending and creasing skin occurs. The microbicidal liquid solution consists of organic polymer dissolved in methylene chloride organic solvent. About 0.60 fl. oz. of solution is held in a glass bottle with a screw cap.

**Preparation:** Wound and surrounding skin should be washed with soap and water and be clean and dry. The area should also be free of creams, lotions or ointments, before applying the liquid solution.

**Application:** The liquid solution should be applied on and around the wound, covering an area that extends at least one and a quarter inches beyond the edges of the wound. Momentary stinging may occur. The clear film forms in less than one minute and should be inspected to be sure complete, continuous covering of the wound and surrounding area has occurred. The dry film covering should be inspected frequently for leakage and loosening of the edges. Although the film itself remains intact under normal use, additional layers may be applied if the dry film begins to slough off or peel around the edges in one to three days – depending on external exposure such as soap and water, rubbing, etc. If it is desired to remove the film, it should be washed gently with soap and water and gently peeled off starting at the outer edges. Following application, the bottle should be tightly capped to prevent evaporation.

**Caution:** For external skin use only. Do not use on deep or infected wounds, or puncture wounds. Contact a physician for deep, puncture or infected wounds, animal bites, redness, swelling, or pain. Intentional inhaling of the contents may be harmful or fatal. Use with adequate ventilation. Keep away from children. Do not use near eyes, mouth, or nose. Avoid contact of the solution with clothes or finished surfaces. Do not store near heat or flames. In case of ingestion, obtain medical assistance immediately. Bottle should be tightly capped after use to prevent evaporation of the solvent.

Ingredients: Organic polymer in methylene chloride organic solvent.

Contents: Glass bottle container with bottle cap. Contains about 0.60 fl. oz. of solution.

Manufactured for Prevent-Plus LLC